

**PHILIPS**  
RESPIRONICS

*Amara View*

Minimal contact

Full face mask

High marks for **performance**,  
no marks on the bridge  
of the nose



\*CPAP and masks ©Philips Respiration

**CPAPXCHANGE**

SLEEP WELL TONIGHT



# Enjoy the view

Amara View has a wider field of vision than Resmed AirFitF20 and Fisher & Paykel Simplus.\*\*

An idea whose time has come. Amara View's innovative design prevents red marks, discomfort or irritation on the nose bridge\*. Amara View is smaller, lighter†, and covers less of a patient's face than Resmed AirFit F20 and Fisher & Paykel Simplus.\*\* It's easy to wear glasses, read, watch TV, and use a computer or tablet before falling asleep.


## Convenient magnetic clips

Amara View, now with magnetic clips, makes attaching or reattaching headgear before bed or in the middle of the night easy, simple and fast.



## Features

- Innovative design prevents red marks, discomfort or irritation on the nose bridge
- Enables wearing glasses, reading, and watching TV
- Easy to use with one modular frame for all cushion sizes
- No bulky cushion and frame in front of your face and eyes

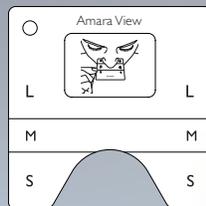

**Amara View's cushion size is determined by measuring the depth of the nose using the simple sizing gauge.**



**Amara View covers less of a patient's face than the Resmed AirFit F20 or the Fisher & Paykel Simplus\*\***

\*Design claim

\*\*Internal testing (May 2014, April 2017, August 2017, January 2018), comparing Amara View with F&P Simplus and Resmed AirFit F20.

† Without quick release tube



# Get started

Call 1 800 345 6743 for more information about how Amara View with magnetic clips can start making a difference for your patients and your business.

Item	Part number	HCPCS code
Small Amara View mask with headgear	1090622	A7030, A7035
Medium Amara View mask with headgear	1090623	A7030, A7035
Large Amara View mask with headgear	1090624	A7030, A7035
Small Amara View mask without headgear	1090632	A7030
Medium Amara View mask without headgear	1090633	A7030
Large Amara View mask without headgear	1090634	A7030

Replacement parts	Part number	HCPCS code
Amara View quick-release tube	1090689	N/A
Small Amara View cushion	1090692	A7031
Medium Amara View cushion	1090693	A7031
Large Amara View cushion	1090694	A7031
Amara View headgear, large	1090696	A7035
Amara View headgear, standard	1090697	A7035
Amara View headgear clip, 2 pk	1090698	N/A
Amara View sizing gauge	1090699	N/A
Amara View magnetic clip, 2 pk	1136626	N/A

# Amara View **fitting guide**

Achieving the right fit with Amara View is easy. Simply follow the instructions below to get started.

**Please read Instructions for Use for further fitting information.**



1

Loosen headgear to a large setting, then disconnect the magnetic headgear clips by gripping and twisting them outward to disconnect from the magnetic headgear clip receiver.



2

The mask will fit against the face straight on, under the nose and over the mouth. Place the nose over, not into, the nostril opening on top of the cushion.

**Note:** Do not place the mask over the nose or place the nose in the cushion nostril opening.



3

Holding the cushion in place against the face, pull the headgear over the head. Push the mask cushion up into position so that the large opening sits over the mouth and the top of the cushion fits under the nose.



4

Connect the magnetic headgear clip to the magnetic headgear clip receiver.



5

The crown strap is very important in achieving a proper seal. Pull the crown strap tab over the top of the head to reduce leaks around the nose.

6



Pull back on the tabs to adjust the top and bottom side straps. The headgear should fit loosely and comfortably. Be careful not to over tighten, which is a common mistake.

7



Connect the flexible tubing (included with the CPAP or bi-level device) to the Amara View quick release tube.

## Dream Family connectivity and compatibility

Amara View connects with the Dream Family, which includes the DreamStation sleep therapy device and DreamMapper patient self-management tool.



Learn more

Find out more about Amara View full face mask. Visit [www.philips.com/amaraview](http://www.philips.com/amaraview)



© 2018 Koninklijke Philips N.V. All rights reserved.  
Specifications are subject to change without notice.

[www.philips.com/respironics](http://www.philips.com/respironics)

Caution: U.S. federal law restricts these devices to  
sale by or on the order of a physician.

RRDPGH EB 5/25/18 MCI 4108250 PN 1122200  
1010 Murry Ridge Lane, Murrysville, PA 15668  
800 345 6443 • 724 387 4000