

# DreamWear cushion **patient fitting guide**

Follow the instructions to get started fitting your nasal or gel pillows cushion

## Nasal cushion

## Gel pillows cushion

1



The cushion opening will be directly under the nostrils. The cushion will hug the nose and leak will be minimal.

**Note:** At no time should the nose be inside the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly.



2



With the mask assembled, place the cushion under the nose.



3



Position the frame on the top of the head. Pull the headgear over the back of the head.



4



To adjust the mask, peel the headgear tabs away from the fabric. Adjust the length of the straps. Press the tabs back against the fabric to reattach.

**Note:** Do not overtighten the headgear.



5



Position the mask until it fits comfortably.



6



When finished, the elbow should rest at the top of the head.

**Note:** Please read the Instructions for Use for further fitting information.



Provided for your reference by

**cpapXchange.com**