MULTI-PATIENT CLEANING INSTRUCTIONS


Your FlexiFit™ 405SL Nasal Mask is provided patient ready. To re-use between patients please follow the guidelines below.

ASSEMBLY / DISASSEMBLY

Your FlexiFit™ 405SL Nasal Mask is fully assembled and ready to use. However, when you need to clean your mask, use the following instructions for reassembly.

REFER TO THE MASK ASSEMBLY DIAGRAM.

1. Place the Foam Cushion (D) into the Mask Base (A).
2. Starting at the apex of the mask, fit the Silicone Seal (C) over the Foam Cushion and onto the Mask Base, finishing at the bottom of the mask.
3. Push the Elbow (B) into the Mask Base.
4. Lay the Headgear (H) flat with the loop side down. Position the assembled mask over the Headgear and attach the four straps into the corresponding slots in the Forehead Rest (F) and the Glider™ Strap (E). This can be done without undoing the Velcro™ tabs, by sliding the Headgear into the slots.

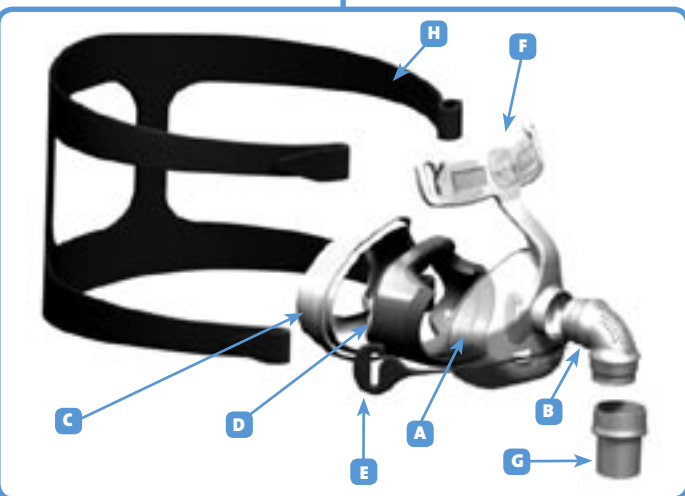CLEANING / DESOILING

- Before disinfection, remove visible soil by disassembling and thoroughly cleaning the mask with a small brush under running potable tap water.

FAILURE TO PROPERLY CLEAN THE DEVICE MAY RESULT IN INADEQUATE DISINFECTION.

HIGH LEVEL DISINFECTION PARAMETERS

- NB: For autoclave cycles, components should be enclosed in a sterilization peel pouch.

MASK FRAME 	AUTOCLAVING Autoclave 15 min exposure at 121°C (250°F). OR THERMAL DISINFECTION Soak parts in hot water bath for 30 minutes at 70°C (158°F). OR STERRAD® 100S Follow manufacturer instructions for Sterrad® 100S Sterilization system. OR CIDEX® OPA Follow manufacturer instructions for high level disinfection. Immerse for 12 minutes at 20°C (68°F). Rinse thoroughly in potable water. OR SPOROX® II Follow manufacturer instructions for high level disinfection. Immerse for 6 hours at 20°C (68°F). Rinse thoroughly in potable water.
SILICONE SEAL 	
ELBOW 	
SWIVEL 	
HEADGEAR 	Hand wash the Headgear in pure soap dissolved in lukewarm water. Air dry out of direct sunlight.
FOAM CUSHION 	Replace with new part.

THE FLEXIFIT™ 405SL NASAL MASK HAS BEEN VALIDATED TO WITHSTAND 20 CYCLES OF THE ABOVE PARAMETERS. DO NOT USE IF BADLY DAMAGED.

Disclaimer: It is the users responsibility to qualify any deviations from the recommended method of reprocessing.

Contact your local Homecare Provider for additional information.

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185043375 Rev C

NOT FOR RESALE

This mask is designed for short term use only. For a long term use mask, please contact your local homecare provider and ask about the Fisher & Paykel Healthcare FlexiFit™ 405 Nasal Mask.

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CPAP X

FLEXIFIT™ 405SL PATIENT INSTRUCTIONS FOR USE

The FlexiFit™ 405SL Nasal Mask is intended for multiple patient or single patient adult use by individuals who have been diagnosed by a physician as requiring CPAP or Bi-Level ventilator treatment.

FITTING YOUR MASK

Remove the plastic cover from the Silicone Seal (C).

1. Holding your mask in one hand, spread the Headgear (H) with your free hand just enough to fit comfortably over your head.
2. Lightly hold the mask against your face covering the nose then slowly pull the Headgear over your head.
3. Attach the Headgear to the Glider™ Strap (E).
4. Pull the lower straps as low as possible down back of neck. Gently tighten straps to remove slack. Connect to CPAP/Bi-Level machine at prescribed pressure.
 - If necessary, tighten the lower straps to eliminate any leaks (Avoid overtightening the straps).
 - Use the Silicone Seal that provides the best fit for you. Small and Large are both provided with your mask.
 - When you need to remove your mask, unhook the Headgear from the Glider™ Strap.

FITTING TIPS

- Forehead pads should rest gently on the forehead without top straps being pulled too tight.
- If there are any leaks after gently tightening the top and lower straps, pull the mask slightly out from the face while straps are still attached. This will allow the Silicone Seal to inflate with air. Put the mask back onto the face.

The FlexiFit™ 405SL and accessories are Latex free. Many people find CPAP therapy causes dryness of the nose and throat. Heated humidification can substantially reduce these symptoms.

OXYGEN/PRESSURE PORT CONNECTOR

If pressure readings and/or additional oxygen are required, an Oxygen/Pressure Port Connector is available (Code 900HC452).

1. Connect the large end of the Connector (T) to the Swivel (J) on the mask and the small end of the Connector to the main breathing tube (R).
2. Open the Port Cap(s) (P) and place the pressure and/or oxygen tube (S) firmly over the Port(s) (Q).

(The two ports are identical; either port may be used for oxygen or pressure measurement. Only remove the Port Cap from Ports that are being used.)

- **Note:** At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, mask selection, and the leak rate.
- **Warning:** If oxygen is used with CPAP/Bi-Level machine, the oxygen flow must be turned off when the CPAP/Bi-Level machine is not operating.
Explanation of Warning: When the CPAP/Bi-Level device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP/Bi-Level machine enclosure. Oxygen accumulated in the CPAP/Bi-Level machine enclosure will create a risk of fire.
- **Warning:** Do not smoke while using or in the vicinity of someone using oxygen with this mask.
- **Warning:** Do not use Polyvinyl Chloride (PVC) Pressure and/or Oxygen tubing. Use silicone tubing instead.
Explanation of Warning: Contact with PVC can cause premature cracking of ports.

TECHNICAL SPECIFICATIONS

EXHAUST FLOW

The FlexiFit™ 405SL Nasal Mask has an air exhaust system that vents the expelled CO₂ through a group of fine holes in the front of the Elbow.

It is important that these holes are not blocked by any object. This controlled leak ensures exhaled CO₂ is expelled from the mask.

PRESSURE (cmH ₂ O)	3	5	7	9	10	11	13	15	17	19	21	23	25
FLOW (L/MIN)	19	25	30	34	35	38	42	44	46	50	54	57	59

RESISTANCE TO FLOW

Pressure drop through mask @ 50 l/min	0.1 cmH ₂ O
Pressure drop through mask @ 100 l/min	0.5 cmH ₂ O

MASK DEAD SPACE

88cc

OPERATING CONDITIONS

- Operating pressure is 3-25 cm H₂O
- Operating temperature range is 5-40°C (40-104°F)

WARNINGS

- This mask should be used only with CPAP or Bi-Level equipment recommended by your physician or respiratory therapist. A mask should not be used unless the CPAP or Bi-Level machine is turned on and operating properly.
Explanation of Warning: CPAP and Bi-Level machines are intended to be used with special masks which have exhalation ports to allow continuous flow of air out of the mask. When the CPAP/Bi-Level machine is turned on and functioning properly, new air from the CPAP/Bi-Level machine flushes the exhaled air out through the mask exhalation ports. However, when the CPAP/Bi-Level machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. Rebreathing of exhaled air for longer than several minutes can in some circumstances lead to suffocations.
- At low CPAP pressures the flow through the exhalation ports may be inadequate to clear all exhaled gas from the tubing. Some rebreathing may occur.
- Do not block the gas exhalation ports or use alternative Diffusers, other than those manufactured by Fisher & Paykel Healthcare. Before use, always ensure that gas is flowing out the exhalation ports or through the Diffuser (if it is being used).
- Do not use the mask if vomiting or feeling nauseous.
- Discontinue using the mask if there is an allergic reaction to any part of the mask. Consult your physician if this occurs.
- If re-using the FlexiFit™ 405SL between different patients, refer to the Multi-Patient Cleaning Instructions for the 405SL Nasal Mask.

CAUTION

- Use the mask for its intended use only, as directed in this booklet.
- US Federal Law restricts this device to sale by, or on order of a physician.

US PATENT NUMBER 6 615 834
Other Patents Pending
FOR ADDITIONAL INFORMATION CONTACT
YOUR LOCAL HOMECARE PROVIDER

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© Fisher & Paykel Healthcare Limited