

Instructions for Set-Up and Use of the



Ion-Distilled Water Filtration System

Be sure to remove all contents from bottle

Filter Set Up

1. Remove a new filter from the sealed plastic bag
Note: Some moisture in the filter bag is normal
2. Soak the unwrapped new filter in cool tap water for 15 minutes
3. Unscrew the red cap from the bottom of bottle and the white cap from the top
4. Set the moist filter, wide end down, into the red cap and screw it securely to the bottom of the bottle
5. Place the bottle, red cap down, on top of the travel tripod
6. Fill the bottle with tap water approximately ½" from the top of the bottle with a glass or cup – Do not fill the bottle directly from a water faucet – the H2O 4 X drip process requires gravity to work most effectively – fast water from a faucet will interfere with filtration
7. Immediately reattach the black cap to the top to avoid leakage and watch the water go through – 400ML of ion-distilled water should take about 5-7 minutes
NOTE: SOME DRIPPING FROM AREAS OTHER THAN THE CENTER HOLE OF THE RED CAP IS NORMAL – ALL WATER IS STILL PASSING THROUGH THE ION-DISTILLATION FILTER
8. For every new filter, allow the water to complete THREE full fill ups and drip-through cycles to safely activate the filter – disposing of the newly filtered water each time

The H2O 4 X system is now ready for daily use

Daily Use

- Fill the bottle as instructed in Steps 6 & 7
- Always empty the bottle completely after each use and store the H2O 4 X bottle containing the moist filter in a cool, dry place when not in use
- Clean the bottle thoroughly at least once a week
- Change the filter every 30 days

Williamson Industries, LLC

Los Angeles, CA 90016

H2O4X.com

Copyright 2025 All Patents Pending

Printed in USA