Figure 1

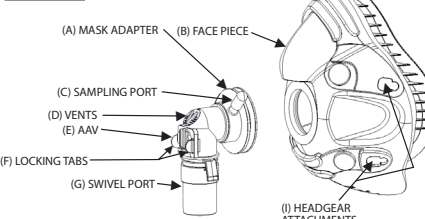
Figure 2

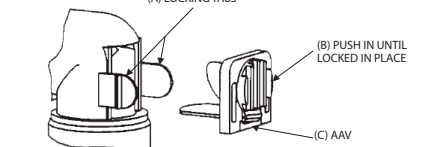
Figure 3

Figure 4

Figure 5


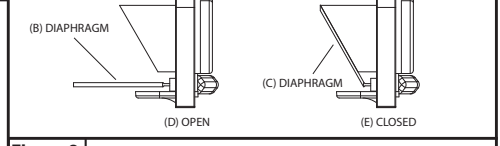
Figure 6

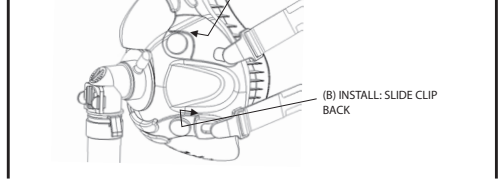
	ESPAÑOL (es)	DEUTSCH (de)	FRANÇAIS (fr)
A	ADAPTADOR MASCARILLA	MASKEADAPTER	ADAPTEUR DE MASQUE
B	PIEZA FRONTAL	GESICHTSTEIL	PIECE FACIALE
C	TOMA MUESTREO	PROBENABNAHMEANSCHLUSS	ORIFICE D'ÉCHANTILLONNAGE
D	ORIFICIOS VENTILACIÓN	LUFTLÖCHER	ORIFICES D'AÉRATION
E	VÁLVULA AAV	AAV	VALVULE VAA
F	LENGÜETAS TRABANTES	EINFRASTZUNGEN	LANGUETTES DE VERROUILLAGE
G	TOMA GIRATORIA	DREHGELENKANSCHLUSS	RACCORDS ARTICULES
H	ACODADA	GEBÖGENER TYP	TYPE COUDÉ
I	CONEXIONES COPIA	KOPFHAUBEN-BEFESTIGUNG	FENTES POUR ATTACHES DU HARNAIS

	ESPAÑOL (es)	DEUTSCH (de)	FRANÇAIS (fr)
A	LENGÜETAS TRABANTES	EINFRASTZUNGEN	LANGUETTES DE VERROUILLAGE
B	PRESIONE HASTA QUE QUEDEN TRABADAS EN SU LUGAR	HINEINDRÜCKEN BIS ES EINSCHNAPPT	ENFONCEZ JUSQU'À CE LA POSITION SONT VERROUILLÉE
C	VÁLVULA AAV	AAV	VALVULE VAA

	ESPAÑOL (es)	DEUTSCH (de)	FRANÇAIS (fr)
A	TOMA AAV	AAV-ANSCHLUSS	ORIFICE VAA
B	DIAPHRAGMA	PLATTENTVILL	DIAPHRAGME
C	DIAPHRAGMA	PLATTENTVILL	DIAPHRAGME
D	ABIERTA	OFFEN	OUVERT
E	CERRADA	GESCHLOSSEN	FERMÉ

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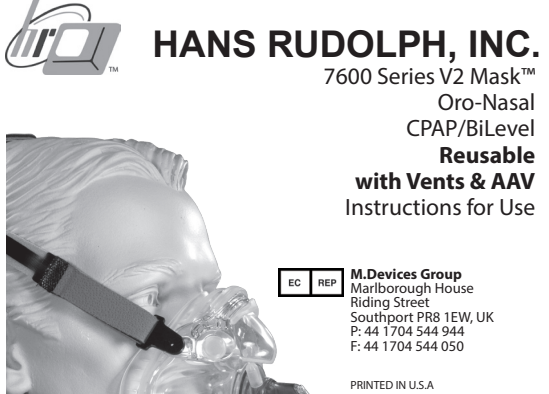CE 0086 © 2015 Hans Rudolph, Inc.

ENGLISH (en) Product Description - Hans Rudolph 7600 Series V2 Mask - Oro-Nasal CPAP/BiLevel Reusable - With Vents and AAV

Intended Use: The 7600 V2 Mask is a reusable, multi-patient, multi-use, Oro-Nasal CPAP/BiLevel mask which incorporates a passive, continuous flow exhaust port. It is intended for use on adults (>30 kg) in homes, hospitals and other clinical settings with CPAP/BiLevel machines for obstructive sleep apnea treatment, and for use with other similar ventilators that use this mask exhaust port configuration providing minimum 3 cm H2O pressure at mask.

Caution: Federal law restricts this device to be used by or on a physician's order.

Warnings:

1. To avoid skin irritation, never use any cleaning solutions or chemicals on this mask or headgear other than a mild detergent and water.
2. Do not block mask vent holes or patient circuit flow.
3. Patients with facial hair may experience mask leakage.
4. Do not use mask with facial hair or patient circuit flow.
5. Do not use with ventilators that require double-limb patient circuits.
6. Do not use with CPAP/BiLevel devices which require separate patient circuit exhaust port.
7. When CPAP/BiLevel device is turned off, or is not operating properly, flow through mask vent holes may be inadequate to clear exhaled gases and some rebreathing may occur which could lead to asphyxiation.
8. Discontinue mask use if irritation or allergic reaction develops due to use.
9. Patient failure to assemble and remove mask after vomiting could result in vomitus aspiration.
10. If supplemental oxygen is used, inhaled percent oxygen will vary depending upon pressure settings, patient breathing pattern, mask size & leak rate.
11. Oxygen flow mask be turned on when CPAP/BiLevel system is not operating.
12. Combination of this mask with devices other than for which mask is intended can alter mask safety and performance becoming unsafe or life-threatening.

Contraindications:

1. Open wounds prone to infection.
2. Hemodynamic or cardiorespiratory instability.
3. Unconsciousness or patients unable to remove mask.
4. Claustrophobia, anxiety, or other discomfort with Oro-Nasal mask.
5. Facial or nasopharyngeal deformity or other inability to fit mask & seal properly.
6. Excessive reflux, GI bleed or other secretions.
7. Impaired cough reflex, nasal hernia, or inability to swallow or clear secretions.
8. Barotrauma, upper airway obstruction or facial trauma.
9. Need for ventilation or ventilator support > 12 hours per day.
10. Recent facial, esophageal, or gastric surgery.
11. Patients under medication with drug that may cause vomiting.
12. Patients requiring immediate intubation.

Risks and Complications:

1. Biosensitivity to device materials (irritation, sensitivity, or other reaction).
2. Irritation/abrasion/ulceration/deformity of face, head, neck, eyes, mucosa, dental).
3. Inhaled, aspirated, applied or handled contaminants (particulates, microbes, body fluids, chemicals).
4. Infection (skin, eyes, mucosa, respiratory).
5. Excessive rebreathed dead space or breathing resistance (dyspnea, atelectasis, barotrauma).
6. Excessive/undesired sound level.
7. Low secretion clearance (especially during upper respiratory infections).

Mask Components (Fig 1):

1. Mask brace with slot openings for slot openings for attachment of headgear strap mounting quick-release clips.
2. Swivel port assembly comprised of sampling port, micro vent holes, anti-siphon valve (AAV) and detachable 22 mm (ISO 5356-1) swivel port.
3. AAV (Fig 2) snaps into swivel port assembly using "locking tabs" for engagement.

Application & Fitting (Fig 4):

1. Determine patient mask size with optional Mask Sizing Gauge (Fig 3).
2. Place mask over patient's nose and mouth (1). Chin should fit into mask chin cup portion with top of mask sealing area on the bridge of the nose.
3. Slide headgear over patient's head (2) with one of the lower quick release clips disconnected. Reconnect headgear strap clip after headgear is placed (3). Bottom straps should be positioned below ears and top straps above ears & below eyes.
4. Adjust headgear Velcro™ strap tension by pulling back slightly on both the top and bottom straps (4 & 5). Adjust straps at crown of head by pulling each in opposing directions until reaching required tension and then reconnecting Velcro™ (6).
5. Connect mask to patient circuit by following ventilator operating instructions.
6. Turn on ventilator. If you detect leaks reposition mask or adjust strap tension. If leak continues, regardless of adjustments, try another mask size.

Safety Features Verification:

1. Open (Fig 5) allows for breathing for patient if ventilator device stops. AAV diaphragm CLOSÉS (>3 cm H2O) the room air port in mask during CPAP/BiLevel ventilation. It OPENS (<3 cm H2O) mask to room air breathing when ventilator device is off. Mask should not be used if AAV is missing, damaged or not functioning properly.
2. Mask Removal with Quick-Release Headgear (Fig 6). To remove mask side one of the lower strap quick-release clips slightly forward towards mask front. It will disconnect from mask; pull mask and headgear to opposite side of up over patient's head.

Disassembly for Cleaning

1. Separate mask from mask.
2. Separate swivel port assembly from mask. AAV from swivel port (squeeze two locking tabs & pull AAV out) and rubber cap plug from mask adapter sampling port. Cleaning Mask piece, swivel port components & headgear.

Mask & swivel port assembly:

1. Soak components for 5 minutes in warm water with mild detergent (neutral pH) wash.
2. Hand wash components with sponge or soft brush; then rinse with warm tap water.
3. Place components in warm water bath and agitate for two minutes.
4. Rinse in clean tap water for at least 1 minute.
5. Allow components to air dry or with clean, lint free cloth.

Headgear:

1. Submerge headgear in warm soapy tap water; gently rub all areas.
2. Rinse in clean tap water for 1 minute or until water is clear and headgear is air dry.
3. Precaution: Do not use bleach, chlorine or alcohol based solutions to clean mask or headgear. These solutions can

damage components. Direct sunlight exposure on mask and headgear components can cause deterioration and reduce product life.

High Level Disinfection (Mask and Swivel port components less Headgear)

1. Thoroughly clean mask prior to disinfection or sterilization
2. For disinfection use Cide™ liquid glutaraldehyde or hot water pasteurization. **Liquid Chemical Disinfection:** Soak the device components in the liquid chemical solution per the manufacturer's instructions. Remove the components and soak in 1500 ml of sterile water for a minimum of 1 minute. Repeat step 2 and then dry the device with a clean lint free cloth (preferably sterile). **Hot Water Pasteurization:** Immerse all device component surfaces in hot water temperatures 71.1°C - 76.6°C for 30 minutes. Dry the device with a clean lint free cloth (preferably sterile).
3. Visually inspect all components for cleanliness, function and defects. If there are any residues, stains or organic debris from repeat the previous steps. Dispose of and replace all defective parts.

Reassembly:

1. Snap AAV into swivel port elbow, ensuring both elbow locking tabs are fully engaged with slots of the AAV.
2. Press rubber cap plug onto sampling port of mask adapter.
3. Install swivel port assembly into face mask grooving operation. The flange of the mask adapter mates with the groove of the mask opening.
4. Mount headgear to face piece with the snap-in quick-release clips.

Functional Check:

1. Turn patient's swivel freely; swivel port assembly is completely engaged in mask face.
2. AAV is installed completely in port with locking tabs fully engaged and the diaphragm flexes freely with no obstruction and functions as described in the safety features section. Do not use if it is non-functional.

Storage

1. Components should be completely dry prior to storage in a sealed poly bag.
2. Label poly bag with disinfection/clean status, mask description, date and initials.

Steam Sterilization (Face Mask assembly less Headgear)

1. Thoroughly clean mask prior to disinfection or sterilization
2. Mask should be completely assembled (less headgear).

Steam Sterilization cycles

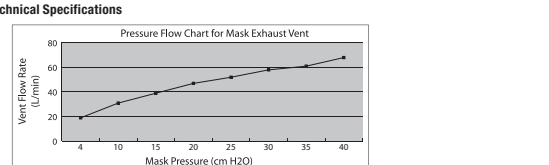
Pre-Vacuum cycle

Gravimetric Displacement cycle

Pre-sterilization: 132.2 ± 3/1° C
Sterilization time: 4 minutes
Dry time: 10 minutes
Packaging: Double pouches or wrapped in CSR

Temperature: 132.2 ± 3/1° C
Duration of the sterilization: 4 minutes
Duration of the cooling: 10 minutes
Storage: Temperature: Below +70 °C, Humidity: 0 – 95% RH
Packaging: Bolsas para esterilización

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Technical Specifications

1. Operating pressure: 4 – 40 cm H2O
2. Mask Resistance to flow: 0.5 cm H2O at 50 L/min, 1.0 cm H2O at 100 L/min
3. AAV Resistance to flow (open port): 3.5 cm H2O at 50 L/min
4. AAV opening/closing pressure: 1.5 - 3 cm H2O
5. Unintentional mask leakage (leak at swivel port connections)
Pressure (cm H2O): 10 20 30
Leak (L/min): 5 8 10
6. Deadspace volume (ml)
L154) M136) S110) XS99) P89)
7. Sound pressure level <25 dBA) measured 15 cm from mask

Mask & Headgear Service Life
Mask and swivel port components are expected to stay in service for minimum of 25 disinfection or steam sterilization cycles or 6 months of use under normal conditions, whichever occurs first. The headgear is expected to stay in service for 6 months of use.

Operational Environment, Mask Assembly & Headgear:

1. Operating: Temperature: 5 – 40 °C, Humidity: 0 – 95% RH
2. Storage: Temperature: Below +70 °C, Humidity: 0 – 95% RH

Recommendations for Disposal:
Treat as conventional solid waste in accordance with local and federal regulations.

Credits:
Velcro™ is a Trademark of Velcro, U.S.A.
Cide™ is a Trademark of Johnson & Johnson Medical Products, Inc.
Tyvek® is a Trademark of DuPont

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