



MEDISTROM


Medistrom 50W Solar Panel Quick Start Guide

1. Unfold the Medistrom 50W Solar Panel and place in direct sunlight.
2. Connect the solar panel to your Pilot-12 Lite or Pilot-24 Lite battery by using the provided cable by plugging the yellow tipped end into the OUT 1 port on the Medistrom 50W Solar Panel.
3. Plug the other end of the cable into the Charge port of your battery.



3 Easy Steps

1. Open panel

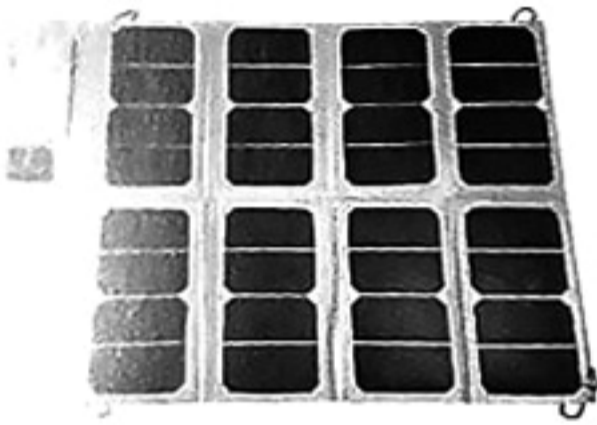

2. Connect cable to panel



3. Connect battery to cable



NOTE: The Resironics DreamStation/S60 cable can be used to charge the Pilot-12 units and the ResMed AirSense 10 cable can be used to charge the Pilot-24 units as well.

For optimal charge times please keep the Medistrom 50W Solar Panel in direct sunlight. Charge times will vary based on cloud cover and angle of direct sunlight. Charging mobile devices via the USB port (OUT 2) while charging the Medistrom Pilot battery will increase battery charge time.

CAUTION! KEEP THE PILOT BACKUP POWER SUPPLY OUT OF DIRECT SUNLIGHT WHEN CHARGING VIA THE 50W SOLAR PANEL.

Keeping the battery in direct sunlight can cause the battery to overheat which can damage the battery and permanently diminish capacity.

Do not use your Pilot battery while it is charging via the 50W Solar Panel.

For more information please contact our technical support team at support@medistrom.com or call 1 888 407 4791