# Instructions for use

## Premium Chin Strap

### Intended Use

The Premium Chin Strap is intended for use by patients >66 lbs. / 30 kg to help prevent mouth leaks that may cause nasal and oral drying during CPAP and bi-level therapy. The chin strap can be used with any Philips Respironics headgear.

⚠ Caution: US federal law restricts this device to sale by or on the order of a physician.

### ⚠ Warnings

- In the event of a power failure, the chin strap will not allow the mouth to open. This will result in rebreathing of exhaled air in the circuit.
- Hand wash prior to first use. Inspect for damage or wear. Discard and replace any components as necessary.
- Some users may experience skin redness, irritation, or discomfort. If this happens, discontinue use and contact your healthcare professional.

### ⊞ Notes

- Any serious incident that has occurred in relation to this device should be reported to Philips and the competent authority of the Member State in which the user and/or patient is established.
- An electronic copy of these instructions can be found at: [www.philips.com/IFU](http://www.philips.com/IFU).

### Cleaning Instructions

Clean your chin strap weekly.

1. Mix liquid dishwashing detergent with warm water (1.3 ml/.04 oz of detergent to 1 liter/33.8 oz of water).


### ⚠ Caution

- Do not use bleach, alcohol, cleaning solutions containing bleach or alcohol, or cleaning solutions containing conditioners or moisturizers.
  - Any deviation from these instructions may impact the performance of the product.
2. Submerge the chin strap and soak in the detergent mixture for 5 minutes.
  3. Agitate the chin strap for another 5 minutes.
  4. Remove the chin strap and rinse thoroughly with 3.8 liters/ 1 gallon of warm water.
  5. Air dry the chin strap out of direct sunlight. Make sure the chin strap is dry before use.
  6. Inspect for damage or wear. Discard and replace as necessary.


### Using the Chin Strap

1. Put on your mask and any cap, strap, or headgear.
2. Close your mouth and place the chin strap over your chin.

⊞ Note: Position the top of the Chin Strap toward the front of your head to apply the appropriate angle of pressure on your chin.

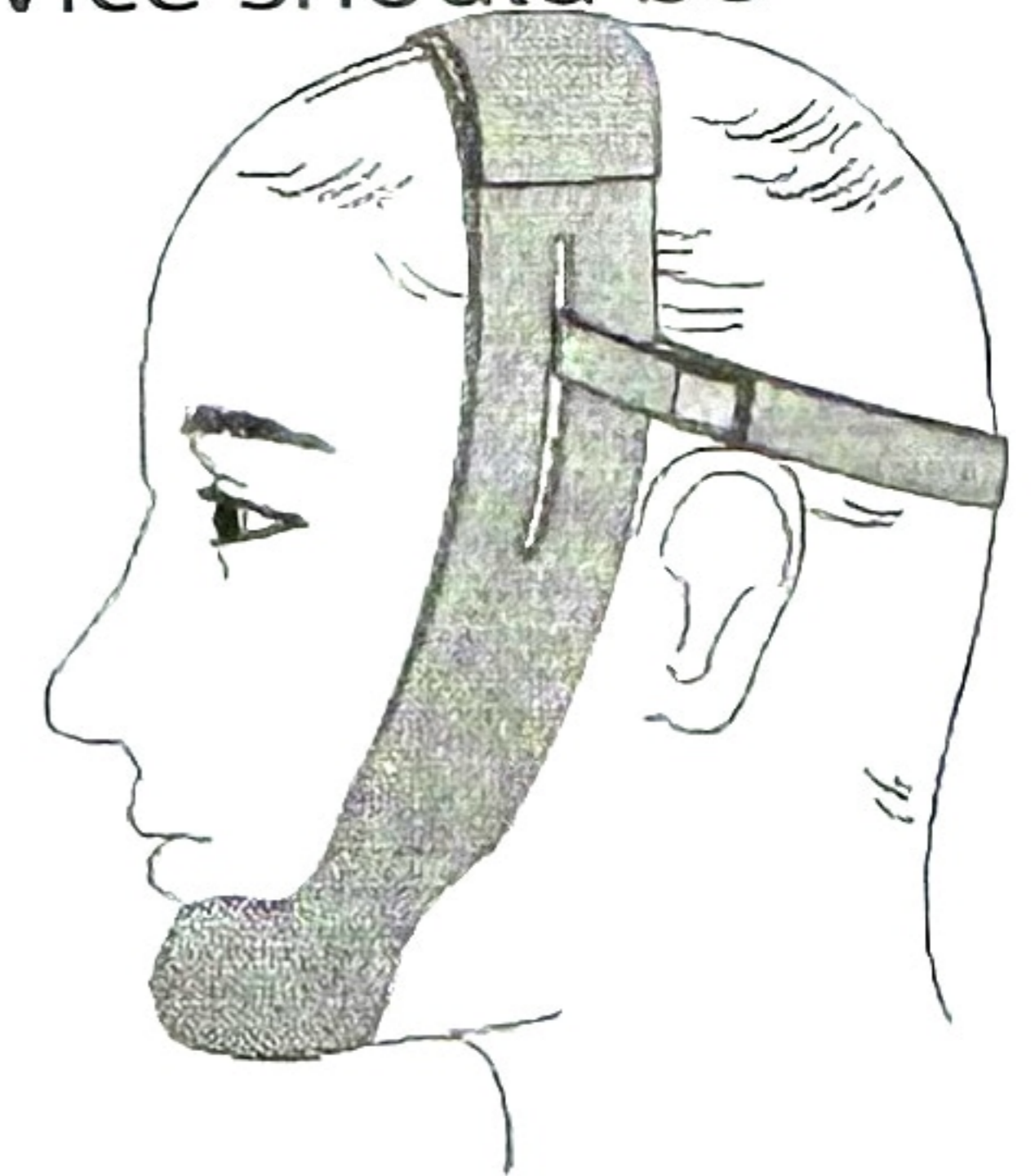⬆ Figure 1

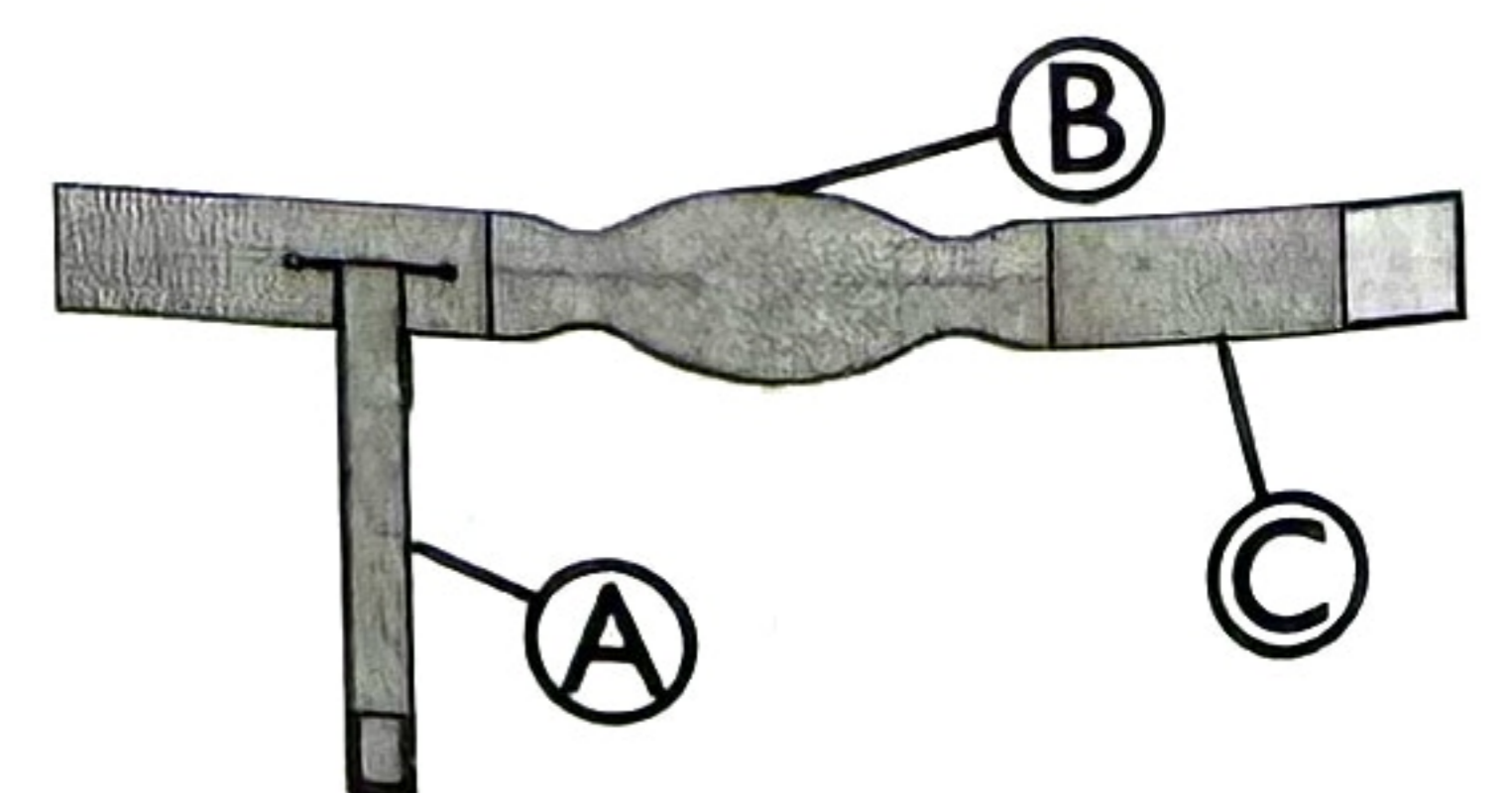⬆ Figure 2

A = Adjustable Support Strap  
 B = Chin Cup  
 C = Chin Strap

3. Gently pull up on both sides of the chin strap and connect the two ends.
-  Note: Avoid creating excessive tension with the chin strap.
4. Pull the adjustable support strap across the back of your head and connect it to the chin strap. Adjust as needed.

### Disposal

Dispose of the chin strap annually

Dispose of this device in accordance with local collections and recycling regulations.

For more information, visit [www.philips.com/recycling](http://www.philips.com/recycling).

### Product Support

Should you experience trouble with this chin strap or require assistance setting up, using, or maintaining the case, please contact your home care provider. If you need to contact Philips Respironics directly, contact customer service at 1-800-345-6443 (USA or Canada only) or +1-724-387-4000, or go to [www.respironics.com](http://www.respironics.com) to find your local customer service contact information.

### Storage Conditions

-4° to +140° F (-20° to +60° C)

15% to 95% RH, non-condensing

### Symbols Glossary

Refer to <http://www.symbols.philips.com> for a description of the symbols used on this device and its packaging.

	Warning or caution - Indicates a warning or caution.		Unique Device Identifier - Indicates the Unique Device Identifier information.
	Packaging unit - Indicates the number of pieces in the package.		Importer - Indicates the entity importing the medical device into the EU.
	Medical Device - Indicates that the item is a medical device.		Single patient - multiple use - Indicates a medical device that may be used multiple times on a single patient.
	Electronic instructions for use - Indicates that relevant information for use of the product is available in electronic form.		Date of Manufacture - to indicate the date on which a product was manufactured. Country of Manufacturer - to indicate the country of manufacture of the product. Note: When applied to the label, "CC" is replaced by the country code.
	Rx Only - Caution! U.S. federal law restricts this device to sale by or on the order of a physician.		

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