


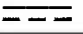



User Manual

Luna G3 DC Power Adapter

Model No.: HT-D15-2450

SKU No.: G3DC

1. Symbols

	Class II Equipment
	Direct current
	Consult instructions for use
	Waste Electrical and Electronic Equipment
IP22	≥ 12.5 mm Diameter, Dripping (15° tilted)
	Manufacturer

2. Description

The Luna G3 DC Power Adapter is intended to provide power for all Luna G3 models, enabling you to operate your device from a vehicle's cigarette lighter socket (12 V DC – 24 V DC power source) in a car, boat, or any other vehicle equipped with a compatible DC power source.

CAUTIONS:

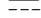
- When using the Luna G3 device with this Luna G3 DC Power Adapter on a moving vehicle, the water chamber should be emptied to prevent water from flowing back into the device and causing damage.
- The Luna G3 device, when equipped with the humidifier, consumes significantly more power compared to using the main device alone. As a result, the battery will drain faster when the humidifier is in use.
- Any serious incidents that occur in relation to this device should be reported to the manufacturer and the competent authority in your country.
- Please take time to read and understand the entire user manual before operating this system. If you have any questions, please contact your equipment supplier.
- The images in this user manual are for reference only; if they differ from the actual product, the physical item takes precedence.
- The expected service life of the Luna G3 DC Power Adapter is five years from first date of use.

3. Specifications

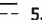
Product Use, Transport and Storage

Operation	Transport and Storage
Temperature: 0°C to 40°C (41°F to 95°F)	-25°C to 70°C (-13°F to 158°F)
Humidity: ≤ 93% Non-condensing	5% to 95% Non-condensing
Altitudes: ≤ 5000 m	≤ 5000 m

DC Input

12 V – 24 V  15 A MAX

DC Output

24 V  5.0 A

Type of Protection Against Electric Shock

Class II Equipment

Degree of Protection Against Ingress of Water

IP22

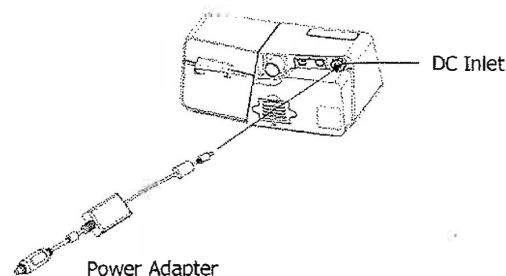
4. Model

Model No.: HT-D15-2450

To order DC Power Adapter, use SKU No.: G3DC

5. Power Connection

- (1) Insert the plug of the power adapter into the DC Inlet of the device.
- (2) Plug the other end of the power adapter into the cigarette lighter socket.



CAUTION:

- Regularly inspect the power adapter cord for any signs of damage. If the cord is damaged, replace the power adapter immediately.
- After a power interruption and subsequent restoration, the device will automatically return to its pre-interruption operating state.

6. Proper Disposal

When the Luna G3 DC Power Adapter reaches the end of its service life, dispose of the device and packaging in accordance with local laws and regulations.

7. Limited Warranty

REACT HEALTH warrants that the device shall be free from defects of workmanship and materials and will perform in accordance with the product specifications for a period of three (3) months from the date of sale by REACT HEALTH to the distributor. If the product fails to perform in accordance with the product specifications, REACT HEALTH will repair or replace, at its option, the defective material or part. REACT HEALTH will pay customary freight charges from REACT HEALTH to the distributor location only. This warranty does not cover damage caused by accident, misuse, abuse, alteration and other defects not related to material or workmanship.

REACT HEALTH DISCLAIMS ALL LIABILITY FOR ECONOMIC LOSS, LOSS OF PROFITS, OVERHEAD OR CONSEQUENTIAL DAMAGES WHICH MAY BE CLAIMED TO ARISE FROM ANY SALE OR USE OF THIS PRODUCT. SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

To exercise the rights under this warranty, contact the local authorized distributors or:

REACT HEALTH

5101 Fruitville Rd., Suite 200
Sarasota, FL 34232
T: (863) 226-6285
For additional information, please visit our website at: www.reacthealth.com.

Manufacturer:

BMC Medical Co., Ltd.

Room 10, 17F, Building 4, Huiya Plaza, No.16 Lize Road, Fengtai District,
100073 Beijing, PEOPLE'S REPUBLIC OF CHINA
Tel: +86-10-51663880
URL: en.bmc-medical.com
E-mail: intl@bmc-medical.com

