




**CPAPXCHANGE**  
SLEEP WELL TONIGHT

**AirFit™ N30i**

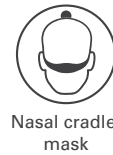

## Fitting guide for AirFit N30i



- Pull the frame and headgear over your head until the elbow is positioned on top and the headgear is behind your head.



- Place the cushion under your nose.



- Tighten or loosen the headgear until the cushion fits comfortably under your nose.



- Squeeze the side buttons on the elbow and detach from the frame.



- Connect the air tubing from your device to the elbow. The mask is now ready to use.



- Reattach the elbow to the frame. The mask is now ready to use.



### Additional options

For patients with longer hair, we recommend feeding their hair through the back of the headgear. If tied in a pony tail, it should sit high enough to fit comfortably through the back of the headgear.

Refer to the user guide for further instructions and complete labeling information including indications, contraindications, warnings and precautions.

1 ResMed Fitting Study of 37 patients comparing AirFit N30i and Philips Respironics DreamWear in 2018.  
2 ResMed Clinical Study of 24 patients comparing AirFit N30i and Philips Respironics DreamWear in 2018.  
Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA +1 858 836 5000 or 1 800 424 0737 (toll free). See ResMed.com for other ResMed locations worldwide. AirFit is a trademark and/or registered trademark of the ResMed family of companies. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip. © 2018 ResMed. 10110761/2 2018-07

[ResMed.com/AirFitN30i](https://ResMed.com/AirFitN30i)



## A more natural mask experience

ResMed AirFit N30i features a top-of-the-head tube design and nasal cradle cushion that allows patients the freedom to sleep in the position of their choice. The SpringFit frame is designed to adapt to each patient's face while only having two frame sizes helps streamline fitting time and SKU management.



[ResMed.com/AirFitN30i](https://ResMed.com/AirFitN30i)



AirFit N30i highlights

**Quick-release elbow** allows for easy disconnection. The pinch-clip elbow stays attached to the tubing to ensure parts do not get stuck or lost.

**SpringFit frame** automatically adjusts to different facial profiles, helping provide an accurate fit the first time.

**Top-of-head design** gives patients the freedom to sleep in any position and closer to their bed partner.

**Soft sleeves** designed for added comfort and reduction in facial marks.

**Curved cradle cushion** is designed to provide a comfortable, stable seal that is less obtrusive.

AirFit N30i Product Codes	
Complete mask systems (frame system, headgear and 2 more cushion sizes)	Product code
AirFit N30i STPK AMER - STD	63800
AirFit N30i STPK AMER - SML	63801
AirFit N30i Sleep Lab Mask STPK - STD	63821
AirFit N30i Sleep Lab Mask STPK - SML	63822
Frame systems (includes frame, cushion and elbow)	
AirFit N30i STD Frame Sys - S	63802
AirFit N30i SML Frame Sys - S	63803
AirFit N30i STD Frame Sys - M	63804
AirFit N30i SML Frame Sys - M	63805
AirFit N30i STD Frame Sys - W	63806
AirFit N30i SML Frame Sys - W	63807
AirFit N30i STD Frame Sys - SW	63808
AirFit N30i SML Frame Sys - SW	63809

Accessories	Product code
AirFit N30i Cushion - SW	63810
AirFit N30i Cushion - M	63811
AirFit N30i Cushion - W	63812
AirFit N30i Cushion - S	63813
AirFit 30i Headgear	63814

Patient study data

In a recent study, the majority of patients preferred AirFit N30i over the current market-leading tube-up mask.<sup>1</sup>

86% | 90%  
found AirFit N30i sealed really well.<sup>1</sup>  
found AirFit N30i very easy to fit and adjust.<sup>1,2</sup>

