INTENDED USE:

Silent Night CPAP Mask Liners are intended to be used with Positive Air Pressure (PAP) medical devices, including but not limited to all CPAP and respiratory masks. The liners should be used as an interface accessory, fitting between the mask and user's face.

CAUTION:

1. Read these *Instructions for Use* completely prior to use.
2. Follow these *Instructions for Use* for proper alignment, attachment, and maximum effectiveness.
3. Do not use for any purpose except as described under **INTENDED USE**.
4. Do not wash, disinfect, or iron.
5. This is a non-sterile medical device.
6. Store unused product in original packaging, in a clean and dry atmosphere. Unused product has no expiration date if stored as described.

WARNING:

1. This product is to be used with intact skin only. Do not use on open wounds or blistered or infected skin.
2. If you experience an allergic reaction, skin irritation, or any adverse reaction while using this product, please discontinue use and immediately contact a healthcare professional.
3. Do not use a single liner for more than eight (8) days.
4. Do not allow children to handle or use this product or its packaging without adult supervision.
5. Do not use if product becomes soiled or contaminated.
6. Dispose of properly after use.
7. Do not expose product to temperatures in excess of 140°F/60°C or lower than -10°F/23°C.

Note: Information in these *Instructions for Use* is subject to change without notice.

© 2023 ATHFAR 828, Inc. All rights reserved.

The *Silent Night*™ logo is a registered trademark of ATHFAR 828, Inc.

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WHAT THIS WARRANTY COVERS: This product is warranted against defects in workmanship and/or materials.

HOW LONG THIS WARRANTY LASTS: This warranty extends only to the original purchaser of the product and lasts for one (1) year from the date of the original purchase.

WHAT ATHFAR 828, INC. WILL DO: ATHFAR 828, Inc. will replace any defective product manufactured by ATHFAR 828, Inc. For all warranty claims, contact ATHFAR 828, Inc. at 231-598-9377 or info@silentnighthealth.com to obtain a return authorization number and additional warranty information.

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Limited 1-Year Warranty | Garantie limitée de 1 an | Garantía limitada de un año | Eingeschränkte 1-Jahres-Garantie