



Provided for your reference by
cpapXchange.com

Simply Fits + Performs



F&P SIMPLUS FULL FACE MASK

When engineering the F&P Simplus our designers set out to create a mask that revolutionized Full Face comfort, seal and easy use.

The F&P Simplus is the result of intensive research and development, many prototypes and thousands of design hours. We consulted patients, equipment providers and clinicians from around the world to develop a mask that **simply fits + performs.**



F&P Simplus Features

The F&P Simplus incorporates three key components, the **RollFit™ Seal**, **ErgoForm™ Headgear** and **Easy Frame**, all designed to work in harmony. In combination, these components offer the comfort, seal and easy use that Fisher & Paykel Healthcare masks are known for.



1. RollFit Seal

As the name suggests, the one-piece seal “rolls” back and forth on the bridge of the nose to adjust automatically. In doing so, the **RollFit Technology** reduces pressure on the bridge of the nose.

The **RollFit Seal** incorporates **Integrated Stability Panels** to enhance the sealing performance and the **Advanced Air Diffuser** is designed for minimal noise and draft.

2. ErgoForm Headgear

This breathable, ergonomically designed headgear self-locates high on the rear of the head, allowing for unrestricted head movement (sideways, up and down).

Incorporating **Stretch and Non-stretch Panels**, the **ErgoForm Headgear** is designed to hold its shape and allow stretch where it is required, for optimal usability and performance.

Embedded VELCRO®* Tabs with **Easy-Clip Hooks** enable fast fitting and removal.

The **Adjustable Crown Strap** enhances mask stability.

3. Easy Frame

This low-profile frame is stable, durable and small, and ensures a clear line of sight.

The one frame fits all three seal sizes and has an **Easy-Clip Frame Attachment** to aid assembly after cleaning.

The **Ball-and-Socket Elbow** rotates for freedom of movement while reducing drag from the CPAP tube.

Mask Reorder Codes		Spare Parts Reorder Codes	
400475	F&P Simplus Mask – Small	400HC579	F&P Simplus Seal – Small
400476	F&P Simplus Mask – Medium	400HC580	F&P Simplus Seal – Medium
400477	F&P Simplus Mask – Large	400HC581	F&P Simplus Seal – Large
		400HC570	F&P Simplus Swivel
		400HC587	F&P Simplus Headgear – Clips & Buckle
		400HC582	F&P Simplus Headgear – Small
		400HC583	F&P Simplus Headgear – Medium/Large
		400HC584	F&P Simplus Mask No H/G – Small**
		400HC585	F&P Simplus Mask No H/G – Medium**
		400HC586	F&P Simplus Mask No H/G – Large**

* VELCRO® is a registered trademark of Velcro Industries B.V.

** Not available in all countries

Manufacturer ■ Fisher & Paykel Healthcare Ltd, 15 Maurice Paykel Place, East Tamaki, Auckland 2013, PO Box 14 348, Panmure, Auckland 1741, New Zealand Tel: +64 9 574 0100 Fax: +64 9 574 0158 Email: info@fphcare.co.nz Web: www.fphcare.co.nz **Australia** Fisher & Paykel Healthcare Pty Limited, 36-40 New Street, PO Box 167, Ringwood, Melbourne, Victoria 3134, Australia Tel: +61 3 9879 5022 Fax: +61 3 9879 5232 **Austria** Tel: 0800 29 31 23 Fax: 0800 29 31 22 **Benelux** Tel: +31 40 216 3555 Fax: +31 40 216 3554 **China** Tel: +86 20 3205 3486 Fax: +86 20 3205 2132 **France** Tel: +33 1 6446 5201 Fax: +33 1 6446 5221 **Germany** Tel: +49 7181 98599 0 Fax: +49 7181 98599 66 **India** Tel: +91 80 4284 4000 Fax: +91 80 4123 6044 **Irish Republic** Tel: 1800 409 011 **Italy** Tel: +39 06 7839 2939 Fax: +39 06 7814 7709 **Japan** Tel: +81 3 3661 7205 Fax: +81 3 3661 7206 **Northern Ireland** Tel: 0800 132 189 **Spain** Tel: +34 902 013 346 Fax: +34 902 013 379 **Sweden** Tel: +46 8 564 76 680 Fax: +46 8 36 63 10 **Switzerland** Tel: 0800 83 47 63 Fax: 0800 83 47 54 **Taiwan** Tel: +886 2 8751 1739 Fax: +886 2 8751 5625 **Turkey** Fisher Paykel Sağlık Ürünleri Ticaret Limited Şirketi, P.O. Box 06371 Ostim, Alinteri Bulvarı 1161/1 Sokak No. 12-14, Ankara, Turkey Tel: +90 312 354 34 12 Fax: +90 312 354 31 01 **UK** Fisher & Paykel Healthcare Ltd, Unit 16, Cordwallis Park, Clivemont Road, Maidenhead, Berkshire SL6 7BU, UK Tel: +44 1628 626 136 Fax: +44 1628 626 146 **USA/Canada** Tel: 1800 446 3908 or +1 949 453 4000 Fax: +1 949 453 4001