



# Ascend™ Full Face

replaceable AIR°gel cushion masks

featuring Advanced Cushion Technology

Rise to a higher level of comfort and convenience with the Ascend full face mask: our first-ever full face mask featuring a replaceable cushion. Our patented AIR°gel replaceable cushion is easily detached and reattached for easy cleaning and cushion replacement, saving both time and money.



## Ascend to a Higher Level of Comfort and Convenience

### Flexible Spacebar™

Unique bendable design provides superior stability, holding a seal without contacting the skin.

### Replaceable Cushion

Now you can replace the cushion without having to replace the whole mask. Available in three sizes.

S M L

### AIR°gel

Maximum comfort, minimum weight. Our patented AIR°gel minimizes pressure points and skin irritation.

### Quick-Release Headgear

Lightweight, breathable headgear slides off easily with one hand.

### Advanced Cushion Technology™ (A.C.T.)

Our ergonomic cushion technology delivers an effective seal for the most comfortable fit.

### 360° Rotating Elbow

For optimal tube positioning and easier mobility

### Active Headgear Connector™

Moves as the patient moves for a reliable seal.



Made in the USA  
of US and imported parts

## What makes our Ascend Full Face Mask different?

- Our easy-to-use replaceable cushion allows you to extend the life of your mask.
- *Flexible Spacebar* is designed to provide superior stability without contacting the skin.
- *Active Headgear Connector* moves when the patient moves to help maintain an effective seal.
- Available in three sizes with simple and breathable quick-release EZFit headgear.

## Advanced Cushion Technology (A.C.T.)

Our patented AIR°gel with an ergonomic design and luxurious feel reduces pressure while maintaining an effective seal.



## Easy-to-Replace Cushion

Disassembly: Just squeeze the sides of the cushion in to release and snap in to replace.



Assembly: Place the top alignment pin in the notch and push the bottom ones in until the pins fit securely.



### Ascend Nasal Masks with EZFit Disposable Headgear Specifications:

Product Number	Component	Size	Mask Weight with cushion	Mask Dimensions with cushion
50825	Mask System & headgear	S, M, & L	NA	NA
50207	AIR°gel Cushion	Small	94.5g	3.92"D x 3.86"W x 7.00"H
50208	AIR°gel Cushion	Medium	103.4g	4.16"D x 4.20"W x 7.00"H
50209	AIR°gel Cushion	Large	107.8g	4.24"D x 4.30"W x 7.00"H
55051	EZFit Headgear	NA	NA	NA

CE 0123



Does not contain natural rubber latex

Also from Sleepnet:

**Ascend™ Nasal**  
replaceable AIR°gel cushion masks



Sleepnet Corporation, 5 Merrill Industrial Drive,  
Hampton, New Hampshire 03842 USA



Scan code to view Ascend Full Face Fitting Tips video.