




1 INCH

Print at 100% (Do not scale or size to fit.)



A. Measure width of nose:

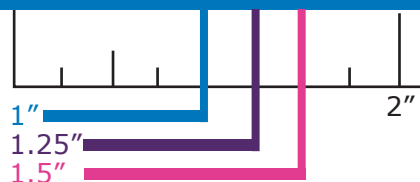

### Nasal Mask Measurement


1. Using the ruler above measure the width of your nose.
2. Using your nose measurement, find your mask size in the chart below.

Nose Size	*Advance	Advance Small	Elan®
Small		●	●
Regular	●		●
Large	●		●

\* SleepWeaver Advance is one size fits most. If your nose measurement falls under 1.25" try SleepWeaver Advance Small first.


### SleepWeaver 3D Neck Sizing Guide

Not actual size  
Use cloth tape measure

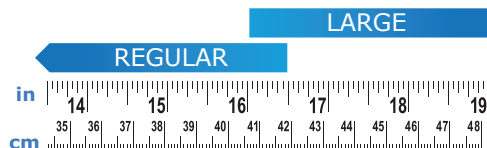


**NOTE: If your measurement falls between sizes on ANY mask, choose the SMALLER mask size.**

A. Measure width of nose:

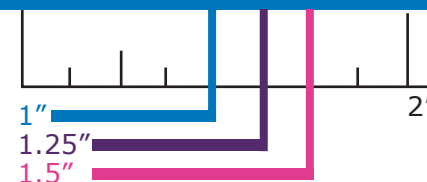


### Full Face Mask Measurements

1. Find measurement A. using top ruler.
2. Find measurement B. using side ruler.
3. Using both measurements, find your mask size in the chart below.

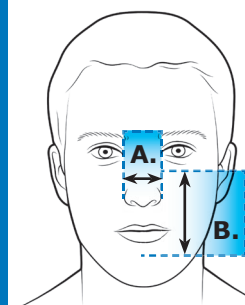

Anew® Size	A. Measurement	B. Measurement
Small	1" to 1.25"	3.25" to 3.5"
Regular	1.25" to 1.5"	3.5" to 3.75"
Large	1.5" & Up	3.75" & Up

B. Measure from bridge of nose to center of chin:

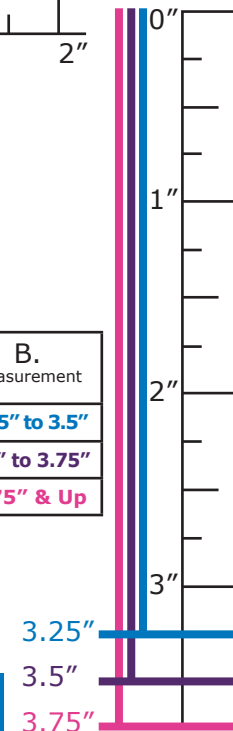

www.circadiance.com +1 888 825 9640 101606 REV3