

| ENG

SoClean

SoClean Neutralising pre-wash

INSTRUCTIONS FOR USE



Use the Neutralising pre-wash to hand wash your PAP equipment. This will neutralise any odours or scents on your PAP equipment.

The neutralising pre-wash is a fragrance-free, hypoallergenic soap and may also be used for periodic cleaning as recommended by the PAP manufacturer.

 soclean.com

 info@soclean.com

 EU 00800 5558 8811

US 866-501-3705


CA 800-916-4577

A000503-14_Rev A 0619


Pre-wash PAP equipment

Disconnect your PAP mask, hose and reservoir from your PAP machine. Before you disconnect your PAP machine to wash these parts, look at where your PAP hose is connected. You will reconnect either the PAP hose or the injection fitting to the same location, depending on your SoClean set-up instructions.

Mix one capful of the Neutralising pre-wash with four litres (1 gallon) of water in a suitable clean container and wash your PAP mask, hose and reservoir.

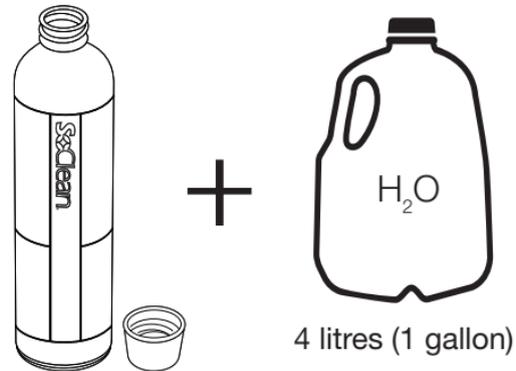


Wash your PAP equipment (mask, hose and reservoir) according to manufacturer instructions. Rinse thoroughly with clean water and let dry.

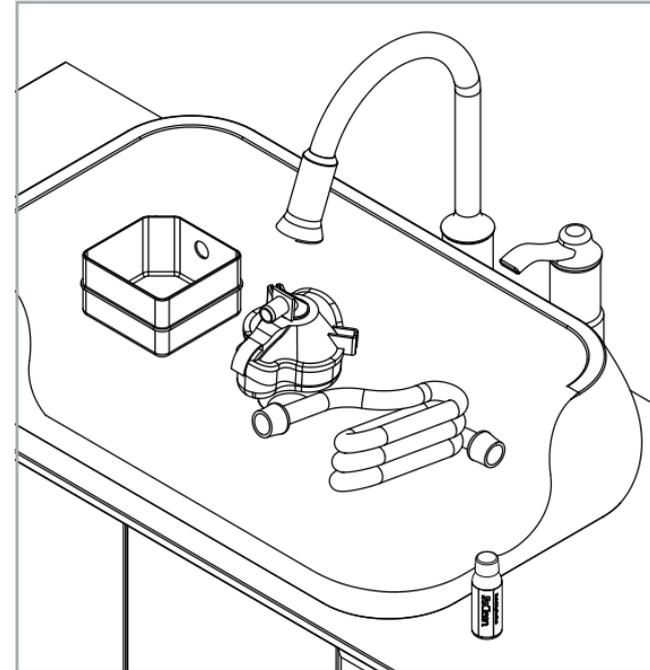

If you have a reservoir, reinsert the reservoir into the PAP machine. Before reattaching the PAP hose, see the set-up instructions in the SoClean device user manual. If your PAP machine requires an adapter, see adapter instructions for set-up.

WARNING:

Do not place Neutralising pre-wash in the PAP reservoir or SoClean disinfecting chamber. Neutralising pre-wash is for cleaning your PAP equipment parts only.

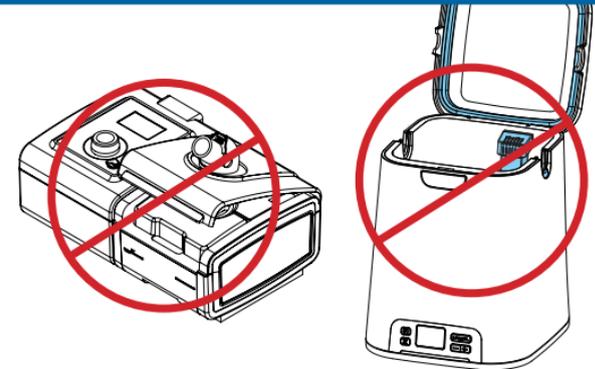

C000504 Rev A