
ResMed

ClimateLineAir™ 11 Oxy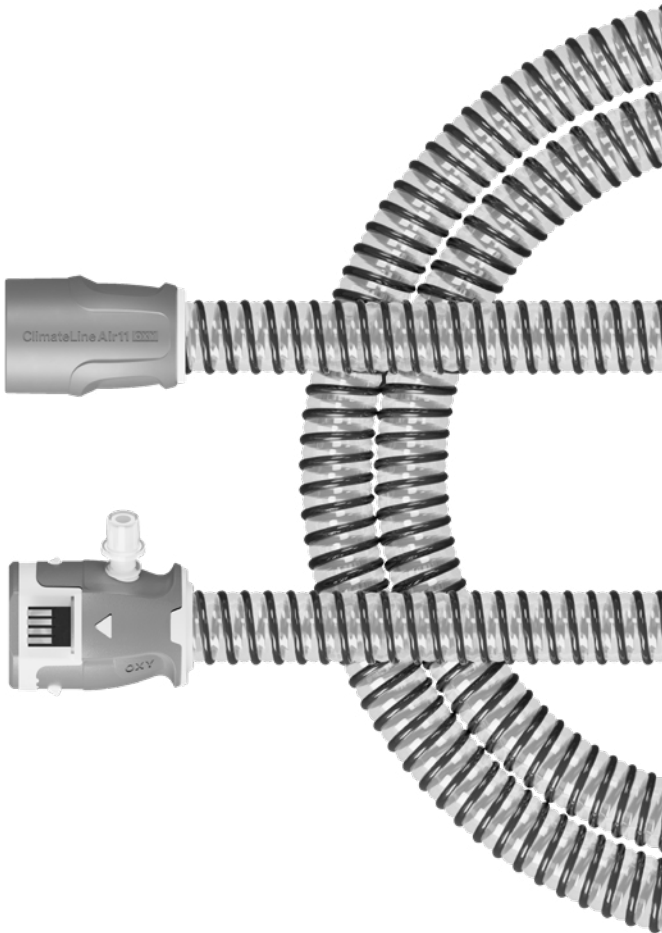

User guide

English | Français | Español | Português

About the ClimateLineAir™ 11 Oxy

⚠ CAUTION

In the US, Federal law restricts this device to sale by or on the order of a physician.

Intended use

The ClimateLineAir 11 Oxy is intended to be used with compatible ResMed devices to deliver heated air to a compatible mask with or without administration of supplemental oxygen.

It is intended for single patient re-use at home and multi-patient re-use in a hospital.


Please read this entire user guide together with your device user guide before using your ClimateLineAir 11 Oxy heated air tubing.

Note:

- The ClimateLineAir 11 Oxy and ClimateLineAir 11 heated tubing are compatible with AirSense™ 11 and AirCurve™ 11 devices.
- The ClimateLineAir 11 Oxy has an integrated oxygen port.
- When used with the device humidifier, the ClimateLineAir 11 Oxy heated air tubing allows you to use the Climate Control feature.

Setting up

To set up the device and tubing: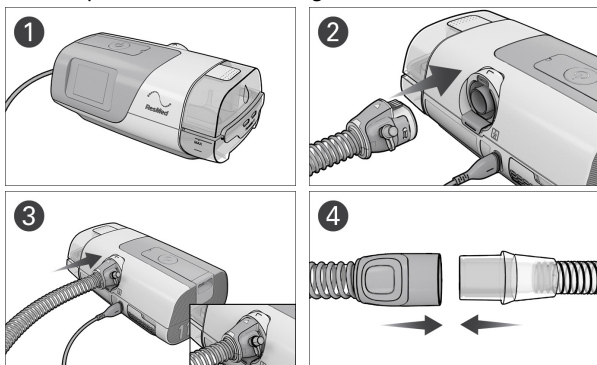

1. Make sure the device is connected to the power supply and turned on.
2. Hold the cuff of the heated air tubing and line up the heated air tubing with the outlet connector at the rear of the device.
3. Push the cuff firmly into place. **ClimateLine™ connected** will briefly display on the device screen.
4. Connect the assembled mask to the free end of the heated air tubing.


Note: The electrical connector end of the heated air tubing is only compatible with the air outlet at the device end and should not be fitted to the mask.

WARNING

The oxygen port should only be used to connect an oxygen tube for supplemental oxygen, and not for any other purpose whatsoever. If supplemental oxygen is not needed, the oxygen port cap should be firmly attached to the oxygen port.

To set up supplemental oxygen:

To set up the ClimateLineAir 11 Oxy with supplemental oxygen, perform the steps outlined in setting up the device and tubing then perform following additional steps.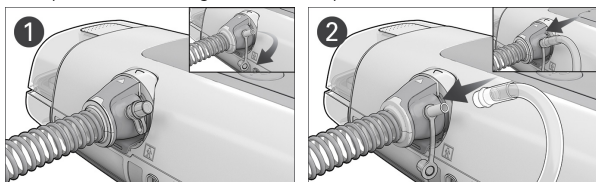

1. Remove the cap from the oxygen connector.
2. Connect the oxygen supply tubing to the oxygen connector. Make sure it is firmly attached

WARNING

Supplemental oxygen must not be used while smoking or in the presence of an open flame.

Notes:

- Before adding oxygen, familiarize yourself with the specific warnings relating to the use of supplemental oxygen. These can be found at the end of your device User Guide.
- If you use the ClimateLineAir 11 Oxy without supplemental oxygen, make sure the cap of the oxygen connector is firmly attached.

Device Settings

Climate Control

Climate Control is designed to make therapy more comfortable by enabling constant temperature and maintaining humidity.

This feature:

- delivers comfortable humidity level and temperature during therapy
- maintains the set temperature and relative humidity during sleep to prevent dryness in the nose and mouth
- can be set to either **Auto** or **Manual**
- is only available when both the ClimateLineAir 11 Oxy and HumidAir™ 11 tub are attached.

Climate Control - Auto setting

Auto is the recommended and default setting. It is designed to make therapy as easy as possible so there is no need to change the temperature or humidity settings.

- Sets the tube temperature to Auto (80°F/27°C). If the air in the mask is too warm or too cold, you can adjust the tube temperature to anywhere from 60 to 86°F (16 to 30°C) or turn it off completely
- Adjusts the humidifier output to maintain a constant, comfortable humidity level of 85% relative humidity
- Protects against rainout (water droplets in the heated air tubing and mask).

Climate Control - Manual setting

Manual is designed to offer more flexibility and control over settings and offers the following:

- Temperature and humidity can be adjusted to find the most comfortable setting
- Temperature and humidity level can be set independently
- Rainout protection is not guaranteed. If rainout does occur, first try increasing the tube temperature
- If the air temperature becomes too warm and rainout continues, try decreasing the humidity.

Note: If Climate Control is set to **Manual**, the **Auto Tube Temperature** setting is not available.

Humidity Level

The humidifier moistens the air and is designed to make therapy more comfortable.

- If you are getting a dry nose or mouth, turn up the humidity
- If you are getting any moisture in your mask, turn down the humidity.
- You can set the **Humidity Level** to Off or between 1 and 8, where 1 is the lowest humidity setting, and 8 is the highest humidity setting.

To update the settings for **Tube Temperature**, **Climate Control**, or **Humidity Level**, tap **MY OPTIONS** from the **Home** screen, go down the list of options, and select the setting. Refer to your device user guide for more information on how to update the device settings.

Cleaning and caring for your tubing


WARNING

- Clean the device and its components according to the schedules shown in this guide, to maintain the quality of the device and to prevent the growth of germs that can adversely affect your health.
- To reprocess this device between patients, follow the cleaning and disinfection instructions in the Clinical guide.

CAUTION

- Do not use bleach, chlorine, or aromatic-based solutions, moisturizing or antibacterial soaps or scented oils to clean the device, the humidifier tub or air tubing. These solutions may cause damage or affect the humidifier performance and reduce the life of the products. Exposure to smoke, including cigarette, cigar or pipe smoke, as well as ozone or other gases, may damage the device. Damage caused by any of the foregoing, will not be covered by ResMed's limited warranty.
- If any visible deterioration of a system component is apparent (cracking, discoloration, tears etc.), the component should be discarded and replaced.
- Ozone or UV light products have not been validated for use with the tube and may lead to discoloration or damage. Always follow cleaning instructions as stated in the User Guide.

Disconnecting

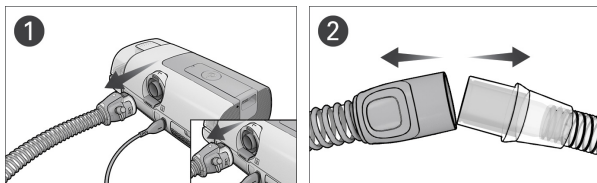1. Pinch the cuff of the air tubing, and gently pull it away from the device.
2. Hold both the cuff of the air tubing and the swivel of the mask, then gently pull apart.

Notes:

- Only hold and pull the cuff of the air tubing. Do not hold or pull the tubing itself as it may cause damage.
- Remove the cap from the oxygen connector and clean separately.

Cleaning

The following instructions are for home cleaning. Instructions for reprocessing devices intended for multi-patient re-use can be found in the clinical guide.

You should clean the air tubing weekly as described.

1. Wash the air tubing in warm water using a household dishwashing liquid. It should not be washed in temperatures higher than 131°F (55°C). Do not wash in a dishwasher or washing machine.
2. Rinse the air tubing thoroughly in water.
3. Allow to dry out of direct sunlight and/or heat.

Checking

You should regularly check the air tubing for any damage. If there are any holes, tears, or cracks, you should replace it.

Reassembly

When the air tubing is dry, you can reconnect it to the device.

1. Connect the air tubing firmly to the air outlet located on the rear of the device.
2. Connect the free end of the air tubing firmly onto the assembled mask.

Note: Replace the dry cap onto the oxygen connector.

Troubleshooting

If you have any problems, have a look at the following troubleshooting topics. If you are not able to fix the problem, contact your care provider or ResMed.

Problem/possible cause	Solution
I am not getting oxygen	
The oxygen connector may be blocked.	Remove the oxygen supply tubing from the oxygen connector and check for foreign material in the port.
Air is leaking from the ClimateLineAir 11 Oxy	
The cap on the oxygen connector may not be attached.	Stop therapy and reattach the cap onto the oxygen connector.
The oxygen connector may be broken	Inspect the oxygen connector. If there are any signs of cracks or damage, replace the ClimateLineAir 11 Oxy. Do not use excessive force when attaching or removing the oxygen supply tubing.

Technical specifications

Environmental conditions	
Operating temperature:	+41°F to +95°F (+5°C to +35°C) Note: The air flow for breathing produced by this therapy device can be higher than the temperature of the room. Under extreme ambient temperature conditions (104°F/40°C) the device remains safe.
Operating humidity:	10 to 95% relative humidity, non-condensing
Operating altitude:	Sea level to 8,500' (2,591 m); air pressure range 1,013 hPa to 738 hPa
Storage and transport temperature:	-13°F to +158°F (-25°C to +70°C)
Storage and transport humidity:	5 to 95% relative humidity, non-condensing
Design life	
Air tubing:	6 months

Air tubing dimensions

Air tubing	Material	Length	Inner diameter
ClimateLineAir 11 Oxy	Flexible plastic and electrical components	6'6" (2.0m)	0.6" (15 mm)

Operating ranges

Maximum recommended pressure:	30 cm H ₂ O (30 hPa)
Temperature range:	+60°F to +86°F (+16°C to +30°C)
Temperature cut-out:	≤ 106°F (≤ 41°C)

Resistance to flow

The table illustrates the resistance to flow of the air tubing:

Air tubing	At flow (l/min) with pressure of 20 cm H ₂ O (20 hPa)	Resistance to flow (cm H ₂ O/l/min) (hPa/l/min) Air tubing only
ClimateLineAir 11 Oxy	30	0.0100
	15	0.0077

Compliance

The table illustrates the compliance of the air tubing:

Air tubing	Compliance (ml/cm H ₂ O [ml/hPa]) with pressure of 60 cm H ₂ O (60 hPa) Air tubing only
ClimateLineAir 11 Oxy	0.500

Note: The manufacturer reserves the right to change these specifications without notice.

Symbols

 Temperature limitation.
  Humidity limitation.

 Operating altitude.
  Atmospheric pressure limitation.

 Manufacturer.
  REF Catalog number.
  DN Device number.

 SN Serial number.
  LOT Batch code.
 Rx Only Prescription only (In the US, Federal law restricts these devices to sale by or on the order of a physician).

 EC REP European Authorized Representative.
  Importer.
  MD Medical device.

See symbols glossary at [ResMed.com/symbols](https://www.resmed.com/symbols).

■ Environmental information

This device should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment.

If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to [ResMed.com/environment](https://www.resmed.com/environment).

Limited warranty

Refer to your device user guide.

Further information

If you require additional information on how to setup, use or maintain the Air11™ system (including ClimateLineAir 11 Oxy heated tubing), or to report unexpected operation or events, please contact the ResMed Service Centre or your care provider.

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© 2025 ResMed. 398053/1 2025-02
See [ResMed.com](https://www.ResMed.com) for other ResMed locations worldwide. Air11, AirCurve, AirSense, ClimateLine, ClimateLineAir and HumidAir are trademarks and/or registered trademarks of the ResMed family of companies. For patent and other intellectual property information, see [ResMed.com/ip](https://www.ResMed.com/ip)
ResMed Pty Ltd