ONE-HANDED OPERATION AID
INSTRUCTIONS FOR USE

Product is non-sterile, not made with BPA or latex.
For use only with PARI Vortex® Non Electrostatic Holding Chamber.
READ ALL INSTRUCTIONS BEFORE USE.


Vortex is indicated for use as a spacer/valved holding chamber for use in delivery of aerosol medication with metered dose inhalers.

ASSEMBLING THE PARTS


Step 1:

- Push the one-handed operation aid onto the inhalation chamber from the back.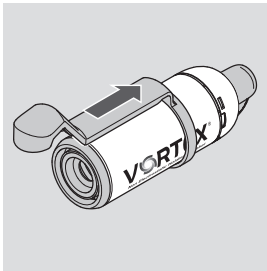


Step 4:

- Position the one-handed operation aid so that its round end encloses the metered dose inhaler. To do this, you may have to push the one-handed operation aid farther forwards or backwards.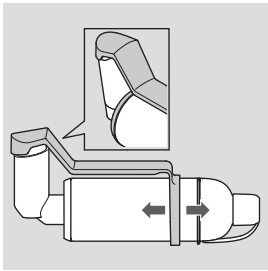

Step 2:


- Remove the cap on the metered dose inhaler.
- Shake the metered dose inhaler vigorously before every use.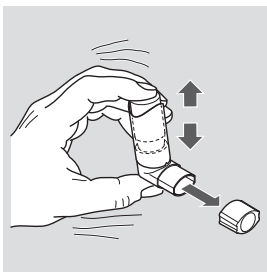

Step 5 (only if using a mask):

Attach the mask (pediatric mask)


- Take the protective cap off the mouthpiece of the VORTEX.
- Attach the mask to the VORTEX mouthpiece.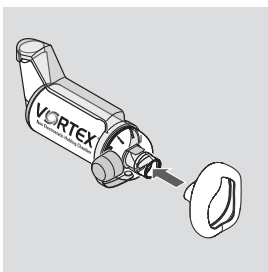

OR


Attach the mask (adult mask)

- Take the protective cap off the mouthpiece of the VORTEX.
- Fit the mask connector into the opening of the PARI adult mask soft.
- Attach the mask to the mouthpiece of the VORTEX with the mask connector.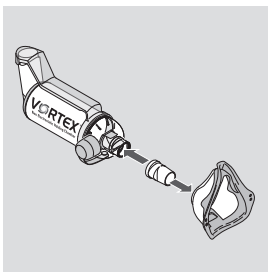

Step 3:

- Insert the metered dose inhaler in the connecting ring on the inhalation chamber.

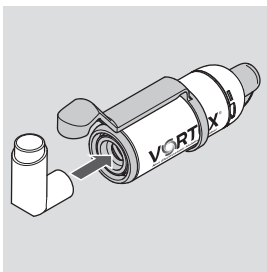INSTRUCTIONS FOR CLEANING

Washing by Hand				
Clean as instructed before first use. After first use, clean and disinfect once a week.	Remove the one handed operation aid from the chamber.	A. Soak part for 15 minutes in luke warm water with liquid detergent. Agitate gently then rinse in clear water. OR B. Place part in top shelf of dishwasher.	A. Rinse in clean water. Shake out in sink. OR B. Remove from dishwasher. Shake out excess water.	Thoroughly air dry on a clean, lint-free cloth or on a drying rack. When dry, store for future use.

DISINFECTION: To disinfect after cleaning, place all parts in boiling water for 5 minutes. Dry as instructed above.

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